concept

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Odsávačka mlieka elektrická

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Elektriskais krūts piena pumpis

Electric breast pump

Elektrische Milchpumpe

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CZ

SK

PL

HU

LV

EN

DE

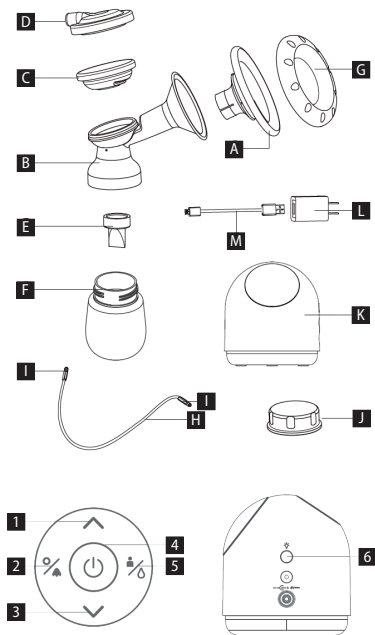
FR

IT

ES


RO

POPIS VÝROBKU
POPIS VÝROBKU
OPIS PRODUKTU
A TERMÉK LEÍRÁSA
IERĪCES APRAKSTS
PRODUCT DESCRIPTION

PRODUKTBESCHREIBUNG
DESCRIPTION DU PRODUIT
DESCRIZIONE DEL PRODOTTO
DESCRIPCIÓN DEL PRODUCTO
DESCRIEREA PRODUSULUI

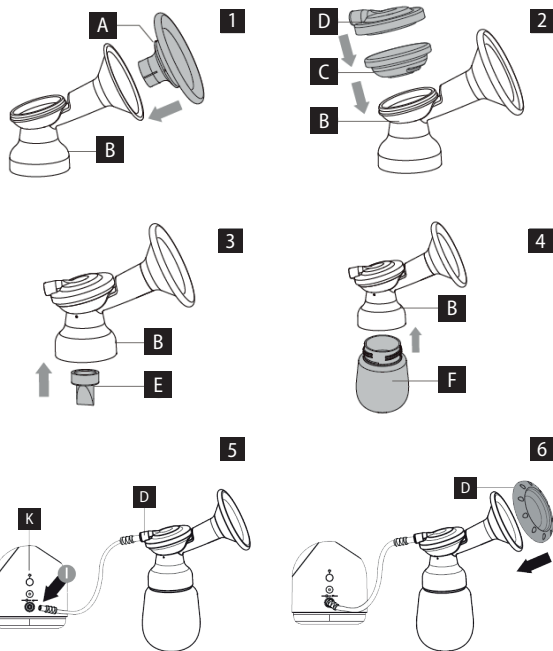**Sestavení odsávačky / Zostavenie odsávačky / Sposób zmontowania laktatora / A mellszívó
 összerakása / Pumpja montáza / Breast pump assembly / Zusammenbau der Milchpumpe /
 Assemblage du tire-lait / Assemblaggio del tiralatte / Montaje del sacaleches / Asamblarea
 pompei de sân**

PODĚKOVÁNÍ

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ACKNOWLEDGMENT

Thank you for purchasing a Concept product. We wish you much joy with your new appliance every day you use it.

Please read the instruction manual carefully before initial use. To refer to this manual any time you need to, we recommend you to keep it in a safe place. And please pass it to any future owner of the appliance.

TECHNICAL SPECIFICATIONS

Input	10 W
Voltage	DC 5 V, 2 A

IMPORTANT SAFETY PRECAUTIONS

- Do not use the appliance in any other way than as described in this manual.
- Remove all the covering and marketing materials from the appliance before the first use.
- Make sure the mains voltage corresponds to the values on the rating plate of the appliance.
- If you need to charge the battery, switch off the product, then connect and use the USB cable and USB power adapter.
- It is forbidden to throw the motor into the fire.
- Do not leave the appliance unattended when it is switched on or connected to the mains.
- When disconnecting the appliance from the wall outlet, never pull the power cord, grasp the plug end and disconnect it by removing it from the receptacle.
- Do not allow children or unauthorized people to handle the appliance, use it out of their reach.
- Do not use the product while taking a shower or a bath.
- Persons with reduced mobility, reduced sensory perception, mental disadvantage or persons not familiar with the operation must use the appliance only under the supervision of responsible person, who is familiar with the instructions.
- Pay extra attention if the appliance is used near children.
- Do not use the appliance as a toy.
- Do not let the power cord touch hot surfaces.
- Do not allow the appliance to hang on the power cord.
- Do not use the appliance in an environment where there are explosive gases and flammable substances (solvents, varnishes, adhesives, etc.).

- Do not use coarse and chemically aggressive substances to clean the appliance.
- Do not use the appliance if it is not working properly, has been dropped, damaged or immersed in liquid. Have it checked and repaired by an authorized service centre.
- Do not use the appliance with a damaged cord or plug and have it repaired immediately by an authorized service centre.
- Do not wrap the power cord around the appliance body.
- Do not use the appliance outdoors.
- The appliance is only suitable for home use and is not intended for commercial use.
- **Do not immerse the power cord, plug or appliance in water or any other liquid.**
- Do not use any accessories other than those recommended by the manufacturer.
- Do not repair the appliance yourself. Contact an authorized service centre.
- This appliance can be used by children aged from 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they have been given supervision or instruction concerning the use of the appliance in a safe way and understand the hazards involved. Cleaning and maintenance performed by the user must not be carried out by children unless they are 8 years old and under supervision. Children aged under 8 years of age must be kept away from the appliance and its power cord. Do not let children play with the appliance.

Damages from failure to follow the manufacturer's instructions are not covered by the warranty.

PRODUCT DESCRIPTION

A	Silicone breast insert
B	Breast shield
C	Silicone against leakage
D	Top cover
E	Valve
F	Bottle (160 ml)
G	Breast shield cover
H	Connecting tube
I	Connector
J	Sealing cover
K	Motor unit with LED touch screen
L	USB adapter
M	USB cable

TROUBLESHOOTING

Pain during use

1. Warm the breasts for five minutes before using the breast pump.
2. Start at the lowest level of suction and then gradually increase.
3. If the above tips have not solved your problems, stop using this product and consult your doctor.

The device is not working.

Check if sufficient power is available, if so, connect a power adapter when using.

Low or no suction power

1. Check all components to ensure they are connected correctly.
2. Make sure that both ends of the tube are properly inserted into the motor unit and top cap.

No milk suction

Ensure that the breasts are aligned with the breast shield with no gaps between them, start pumping using a low suction level.

Panel layout

1	Button to increase suction
2	Nasal suction
3	Suction reduction button
4	ON/OFF button
5	Stimulation/suction mode
6	Night light

CLEANING AND MAINTENANCE

After use, wipe the motor with a clean damp cloth.

Do not clean the motor and adapter in water.

Except for the motor and adapter, you can put the other parts in the sterilizer.

When assembling and disassembling the breast pump, make sure that all components are assembled correctly.

Do not clean the base by immersing it in water. We recommend wiping it only with a clean cloth.

SERVICE

Extensive maintenance or repair, which requires access to internal parts of the product, has to be performed by a qualified service centre.

ENVIRONMENTAL CONCERNS

- Packaging materials and old appliances should be recycled.
- Packaging materials may be disposed of as sorted waste.
- Dispose the plastic bags made of polyethylene (PE) of as sorted waste.

Recycling of the appliance at the end of its lifespan

This appliance is labelled in accordance with European Directive 2012/19/ EU concerning used electrical and electronic appliances (Waste Electrical and Electronic Equipment - WEEE). The symbol on the product or on its packaging indicates that this product may not be treated as household waste. Instead, it should be taken to the appropriate collection point for the recycling of electrical and electronic equipment. By ensuring this product is disposed of correctly, you will help prevent potential negative consequences for the environment and human health, which could otherwise be caused by inappropriate waste handling of this product. The disposal of such material has to be done according to the recycling regulations. For more detailed information about recycling of this appliance see the authorized local office or service for household waste disposal or the shop, where you purchased the appliance.

The product meets all the necessary requirements of the EU directives applicable to the specific product.

Changes in text, design, and technical specifications are subject to change without notice and we reserve the right to change them.

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